


FluoreGuard Mounting Medium (Hard Set)

Description: FluoreGuard Mounting Medium (Hard Set) is a water-soluble medium designed for semi-permanent coverslipping of fluorescent slide preparations. FluoreGuard dries to a semi-rigid layer which eliminates tissue damage due to moving coverslips and facilitates long term storage. The (Hard Set) formulation provides better drying in humid conditions.

Uses/Limitations:

- Not to be taken internally.
- For In-Vitro Diagnostic use.
- Histological applications.
- Do not use if reagents become cloudy.
- Do not use past expiration date.
- Use caution when handling reagents.
- Non-Sterile.

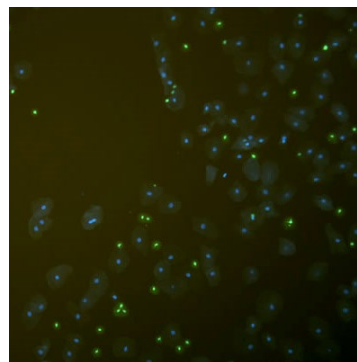

Ordering Information and Current Pricing at www.scytek.com

Availability:	<u>Item #</u>	<u>Volume</u>
	FMH030	30ml Dropper
	FMH060	60ml Dropper
	FMH500	500ml
	FMH999	1000ml

Storage: Store at 18-25°C.

Precautions:

- Avoid contact with skin and eyes.
- Harmful if swallowed.
- Follow all Federal, State, and local regulations regarding disposal.

Procedure:

1. Apply several drops of FluoreGuard Mounting Medium (Hard Set) and coverslip.
2. Allow FluoreGuard to dry for 30 minutes prior to viewing. Product will dry to a semi-solid form overnight.
3. FluoreGuard is an aqueous based medium and may be removed by soaking in PBS overnight.

References:

1. Wanninger S, Donati M, Di Francesco A, Hässig M, Hoffmann K, Seth-Smith HM, Marti H, Borel N. Selective Pressure Promotes Tetracycline Resistance of Chlamydia Suis in Fattening Pigs. PloS one. 2016 Nov 28;11(11):e0166917.

Storage: 18°C
 25°C

 ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 U.S.A.

CE IVD

EC REP

Emergo Europe
 Westervoortsedijk 60
 6827 AT Arnhem, The Netherlands