Trajan microscope slides

Series 2 Enhanced Adhesive

Intended use

Trajan Series 2 Enhanced Adhesive microscope slides are for professional use only. They are recommended for In Vitro Diagnostic (IVD) applications, in particular for mounting Formalin Fixed Paraffin (FFPE) tissue sections to be used in immunohistochemistry (IHC) staining techniques.

Series 2 Enhanced Adhesive microscope slides are hydrophobic in nature and designed to achieve immediate tissue adhesion.

For other applications, please validate before use.
For use with ESPO and ESPO II Path slide printers.

Instructions for FFPE tissue sections

1. Cut the paraffin embedded tissue blocks at a thickness of 2-5 μm on the microtome. Paraffin will make the sections stick together to form a ribbon.
2. Separate the ribbon from the blade edge at the microtome with a pair of forceps or tweezers and place across the surface of a warm, distilled water bath. Recommended water temperature 40-45°C, or 50-52°C for hard paraffin. Water bath additives are not recommended.
3. Individual specimen sections can be detached whilst floating on the water with gentle pressure from the tips of forceps or tweezers.
4. Confirm that the specimen section has no wrinkles or extension.
5. Hold the slide at an angle, contact the tissue section's end in the water bath with the dipped end of the slide, and skim the tissue section onto the slide.
6. Once the end or center of the tissue section is on the slide, wait for the water between the tissue and the surface of the slide to wick off. This will ensure complete contact between the specimen section and the surface of the slide.
7. Skim the tissue section completely onto the slide to adhere. Position the specimen carefully as bonding will occur quickly.
8. Air dry the slides in a vertical position for a minimum of 10 minutes to ensure the section is firmly attached.
9. Dry the slides in an oven at 40°C overnight or at 60°C for 1 hour.

Storage and handling

- Store the slides in original packaging, in a dry environment, preferably protected from sources of heat and light.

- Store at room temperature and below 75% humidity. Recommended storage temperature 2-30°C.
- Avoid large variations in temperature conditions during storage as excessive cooling can lead to condensation on the slides.
- Where large temperature variations may occur during storage, stabilize slides at room temperature before opening the packaging.
- Oldest slides should be used first.
- Do not use after expiry date.
- Packaging provides protection against environmental factors, however if the product is stored under conditions other than those specified, they must be verified by the user.
- Please contact your regional Trajan representative if suboptimal performance is noticed.

Important

⚠ Expiry date

The expiry date for Trajan Series 2 Enhanced Adhesive microscope slides is the last day of the indicated month, which is printed on the base of the slide pack, when stored at temperatures between 2 and 30°C and humidity below 75%.


⚠ Caution

- Single use – do not reuse.
- Beware of sharp glass edges.
- Trained personnel and professional use.
Do not handle slides with bare hands, or touch the glass surface, hold by the edges.
- Tissue loss can be prevented by complete drying overnight at 40°C, or at least 30 minutes at 60°C, as determined by user.

Information and support

Visit www.trajanscimed.com or contact techsupport@trajanscimed.com

Specifications are subject to change without notice.

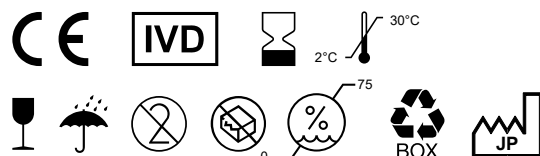	Trajan Scientific Australia Pty Ltd (Sponsor) 7 Argent Place, Ringwood, VIC 3134, AU Tel: +61 (0) 3 9874 8577
EC REP	Axel Semrau GmbH Stefansbecke 42, 45549 Sprockhövel, Germany
US AGENT	Trajan Scientific Americas Inc 1421 W Wells Branch Parkway, Suite 108, Austin TX 78660, USA Tel: +1 512 837 7190

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