

Prostate Specific Acid Phosphatase (PSAP) Clone PASE/4LJ

Catalog Number	Format	Volume
A00027-0002	(Ready-To-Use)	2 ml
A00027-0007	(Ready-To-Use)	7 ml
A00027-0025	(Ready-To-Use)	25 ml

Intended Use

For In-Vitro Diagnostic Use. This antibody is intended for the qualitative visualization of the anatomical elements listed in the Specificity section. It is intended to be used within an Immunohistochemistry (IHC) procedure on formalin-fixed paraffin-embedded (FFPE) human tissue followed by visualization by light microscopy.

Description

Titer/Working Dilution: Ready-to-Use: No further dilution required.

Species: Mouse
Immunogen: Recombinant human wild type p53 protein.
Clone: DO-1
Isotype: IgG2a
Format: Ready-To-Use antibody has been pre-titered and quality controlled to work on formalin-fixed paraffin-embedded as well as acetone fixed cryostat tissue sections. No further titration is required.

Specificity: This antibody reacts with an amino terminal epitope (residue 37-45) of human wt and mutant p53. p53 has been classified as a tumor suppressor gene or anti-oncogene. Presence of excess amounts of the mutant form of the protein can lead to functional inactivation of wild type p53. Mutations and allelic loss of the p53 gene have been associated with malignant transformation in a wide variety of human tumors.

Species Reactivity: Human, Bovine, Monkey. Reacts slightly with Mouse and Rat. Others not tested.

Positive Control: Colon Carcinoma

Cellular Localization: Nuclear

Microbiological State: Nonsterile

Materials and Reagents Required but not Provided

- Control tissue and reagents
- Xylene, graded alcohols, and deionized/distilled water
- Antibody Diluent.
- IHC detection system. Suggested: ScyTek Cat# ABZ125 "CRF Anti-Polyvalent HRP Polymer" and ScyTek Cat# ACV500 "DAB Chromogen/Substrate Kit (High Contrast)".
- Wash buffer for rinses (ScyTek Cat# TBT500)
- HIER Retrieval Solution
- Hematoxylin counterstain and bluing reagent (ScyTek Cat# HMM500 and BRT500)
- Mounting medium and coverslips

Note: ScyTek Laboratories has a wide range of IHC reagents and ancillaries that can be found at scytek.com.

Procedure

1. **Tissue Section Pretreatment (Highly Recommended):** Staining of formalin fixed paraffin embedded tissue sections is significantly enhanced by pretreatment with Tis-EDTA HIER Solution (10x) pH 9.0 (ScyTek catalog# TES500) or Citrate Plus (10x) HIER Solution (ScyTek catalog# CPL500)

2. **Primary Antibody Incubation Time:** We suggest an incubation period of 30 minutes at room temperature. However, depending upon the fixation conditions and the staining system employed, optimal incubation should be determined by the user.

3. **Visualization:** For maximum staining intensity we recommend the "Retrieval HRP Anti-Polyvalent Lab Pack" (ScyTek catalog# RPL125, see IFU for instructions) combined with the "DAB Chromogen/Substrate Bulk Pack (High Contrast)" (ScyTek catalog# ACV500, see IFU for instructions).

Storage and Stability

Do not Freeze. Store at 2-8°C. Return to 2-8° immediately after use. Do not use after expiration date printed on label. Verify visually that antibody has not been contaminated before use. Do not use if reagent becomes cloudy or precipitates.

Limitations

Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining can cause inconsistent results. Variations in fixation and embedding or the inherent nature of the tissue specimen may cause variations in results. Endogenous peroxidase activity or pseudoperoxidase activity in erythrocytes and endogenous biotin may cause non-specific staining depending on detection system used. This data sheet's recommendations and procedures were validated using ScyTek IHC reagents and may not be suitable for other detection systems.

Precautions


- Contains Sodium Azide as a preservative (0.09% w/v), do not ingest. Sodium Azide may react with lead and copper plumbing to form highly explosive metal azides. Upon disposal, flush with large volumes of water to prevent azide build-up in plumbing. This product contains no hazardous material at a reportable concentration according to U.S. 29 CFR 1910.1200, OSHA Hazardous Communication Standard and EC Directive 91/155/EC.
- Do not pipette by mouth.
- Avoid contact of reagents and specimens with skin and mucous membranes.
- Avoid microbial contamination of reagents or increased nonspecific staining may occur.
- The user must validate any procedures and recommendations that differ from this data sheet.
- The SDS may be found at scytek.com


References

- Nigro et al. Nature 342: 705, 1989.
- Cattoretti et al. Int J Cancer 41: 178, 1988.
- Bartek et al. Oncogene 6: 1699, 1991.
- Malkin et al. Science 250: 1233, 1990.

Warranty

No products or "Instructions For Use (IFU)" are to be construed as a recommendation for use in violation of any patents. We make no representations, warranties or assurances as to the accuracy or completeness of information provided on our IFU or website. Our warranty is limited to the actual price paid for the product. ScyTek Laboratories, Inc. is not liable for any property damage, personal injury, time or effort or economic loss caused by our products.

Storage: 2° C  8° C



ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 U.S.A.




Emergo Europe
 Westervoortsedijk 60
 6827 AT Arnhem, The Netherlands