


## Van Gieson's Solution

**Description:** The Van Gieson's Solution is intended for use in the histological visualization of collagen in tissue sections. This reagent is used in many special stain procedures including the Colloidal Iron Stain Kit and the Elastic Stain Kit (Modified Verhoeff's).

|                |        |
|----------------|--------|
| Collagen:      | Red    |
| Muscle Fibers: | Yellow |
| Cytoplasm:     | Yellow |

**Uses/Limitations:** Not to be taken internally.  
 For In-Vitro Diagnostic use only.  
 Histological applications.  
 Do not use if reagents become cloudy.  
 Do not use past expiration date.  
 Use caution when handling reagents.  
 Non-Sterile.

**Control Tissue:** Lung  
 Uterus  
 Muscle  
 Kidney

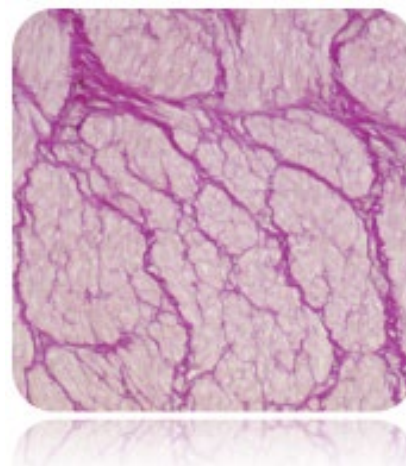

### Ordering Information and Current Pricing at

[www.scytek.com](http://www.scytek.com)

| Availability: | Item # | Volume  |
|---------------|--------|---------|
|               | VGS125 | 125 ml  |
|               | VGS500 | 500 ml  |
|               | VGS999 | 1000 ml |

**Storage:** Store at 18-25°C.

**Precautions:** Avoid contact with skin and eyes.  
 Harmful if swallowed.  
 Follow all Federal, State, and local regulations regarding disposal.

### Procedure:

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Apply adequate Van Gieson's Solution to completely cover tissue section and incubate for 1 minute.
3. Rinse slide using absolute alcohol.
4. Dehydrate in 2 changes of absolute alcohol, clear, and mount in synthetic resin.

Storage: 18°C  
 25°C

 ScyTek Laboratories, Inc.  
 205 South 600 West  
 Logan, UT 84321  
 U.S.A.

CE IVD

EC REP  
 Emergo Europe  
 Westervoortsedijk 60  
 6827 AT Arnhem, The Netherlands