

Instructions For Use ZFN-IFU

Rev. Date: Feb 18, 2025

Revision: 4

Page 1 of 1

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Zinc Formalin Solution

Description: Zinc Formalin Solution is a tissue fixative for routine procedures and in various special stain procedures.

Uses/Limitations: For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling these reagents.

Availability/Contents:

<u>Item #</u>	<u>Volume</u>
ZFN125	125 ml
ZFN500	500 ml
ZFN999	1000 ml

pH: 5.2 (Adjust if desired).

Storage: Store at room temperature.

Procedure (Standard):

1. Trim specimen to smallest practical size.
2. Cover tissue with an amount of fluid equal to 20 times the specimen volume.
3. Fixation times range from 5 to 48 hours depending on specimen size.

Precautions: Poison. Do Not Take Internally.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.



IVD

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands