# BIO TAL



Clinisciences Froup



# Precision Identity Tracking in NGS: Unveiling SNPXplex®'s Potential


<u>SNPXplex®</u> technology enables the simultaneous genotyping of 15 SNPs (single nucleotide polymorphisms) using an allele-specific fluorescent multiplex PCR technique that's quick and easy to use. The results obtained can be compared with NGS genotyping results for the same polymorphisms to ensure identity verification.

The kit **SNPXplex®** can also be used for sex determination, reinforcing its clinical utility.

<u>SNPXplex®</u> technology represents a major advance in the field of identity vigilance applied to high-throughput sequencing tests. Providing molecular geneticists with a reliable, proven solution enables them to apply international best practice recommendations and helps reduce the time taken to deliver results to patients.

# **Storage**

- Reagents can be stored at -20°C.
- Reagents can be stored at room temperature for up to 8 days, protected from light.

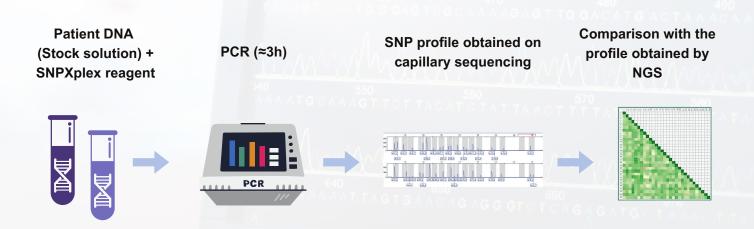

The technique is suitable for routine use. In fact:


- It is fast -a single PCR is performed from the DNA in the mother tube, followed by deposition on a capillary sequencer.
- · It is inexpensive.
- It uses equipment available in any molecular genetics laboratory.

Distributed by:

CliniSciences Group



- PCR pre-mixes are stable for at least 12 months.
- The SNPs chosen are in genes not involved in any pathology: there is no risk of incidental findings linked to NGS sequencing of the regions containing the SNPs.
- The exact combination of SNPs can be used in all custom capture kits: thus, the SNPXPlex technique can be the only barcoding technique in a laboratory using different NGS panels.
- The informativeness of the combination of SNPs is such that, in a series of 96 patients, the risk of two patients presenting the same SNPs profile is < 5%, whatever the origin of the patients, this risk reaches 0.2% for patients of European origin.
- Interpreting SNPXPlex results is quick and easy.

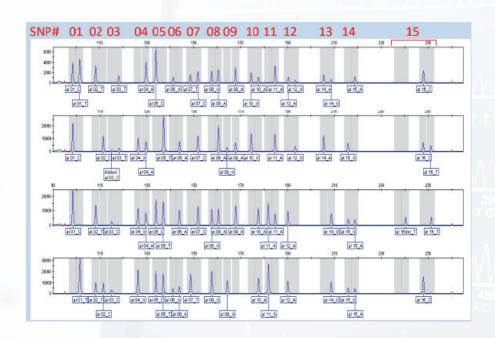

- NGS SNP genotyping results are reliable (>30X coverage).
- A customized Excel file facilitates rapid comparison of SNP profiles obtained via NGS and the SNPXPlex Kit for a series of up to 96 patients.
- This makes it possible to determine whether several patients in the series share the same SNPs profile.
- The SNPXPlex technique is also applicable to exome and genome results, with coverage of 15 SNPs proving adequate.
- If the SNP profiles obtained by NGS and SNPXPlex are perfectly identical, and no
  patient in the series shares the same SNP profile, there's no need to check the NGS
  sequencing results by another method (subject to the quality of the NGS
  sequencing).
- For any patients in the series with an identical SNP profile, validation using an appropriate method will be required.

Distributed by:

# CliniSciences Group

### Austria

Company: CliniSciences GmbH Address: Sternwartestrasse 76, A-1180

Wien - Austria Telephone: +43 720 115 580 Fax: +43 720 115 577

Email: oesterreich@clinisciences.com Web: https://www.clinisciences.com



# Finland

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: suomi@clinisciences.com Web: https://www.clinisciences.com

# Iceland

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: island@clinisciences.com Web: https://www.clinisciences.com

# Netherlands

Company: CliniSciences B.V. Address: Kraijenhoffstraat 137A 1018RG Amsterdam, Netherlands Telephone: +31 85 2082 351 Fax: +31 85 2082 353

Email: nederland@clinisciences.com Web: https://www.clinisciences.com

# **Portugal**

Company: Quimigen Unipessoal LDA Address: Rua Almada Negreiros, Lote 5, Loja 14, 2615-275 Alverca Do Ribateio - Portugal

Telephone: +351 30 8808 050 Fax: +351 30 8808 052

Email: info@quimigen.com Web: https://www.quimigen.pt

# Switzerland

Company: CliniSciences Limited Address: Marktgasse 18 8302 Kloten -

Switzerland

Telephone: +41 (044) 805 76 81 Fax: +41 (044) 805 76 75

Email: switzerland@clinisciences.com Web: https://www.clinisciences.com

### Belgium

Company: CliniSciences S.R.L Address: Avenue Stalingrad 52, 1000

Brussels - Belgium Telephone: +32 2 31 50 800 Fax: +32 2 31 50 801

Email: belgium@clinisciences.com Web: https://www.clinisciences.com

### France

Company: CliniSciences S.A.S Address: 74 Rue des Suisses, 92000

Nanterre- France

Telephone: +33 9 77 40 09 09 Fax: +33 9 77 40 10 11 Email: info@clinisciences.com Web: https://www.clinisciences.com

# Ireland

Company: CliniSciences Limited

Address: Ground Floor, 71 lower Baggot street Dublin D02 P593 - Ireland

Telephone: +353 1 6971 146 Fax: +353 1 6971 147

Email: ireland@clinisciences.com Web: https://www.clinisciences.com

# Norway

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: norge@clinisciences.com

Web: https://www.clinisciences.com

# Spain

Company: CliniSciences Lab Solutions Address: C/ Hermanos del Moral 13 (Bajo E), 28019, Madrid - Spain Telephone: +34 91 269 40 65 Fax: +34 91 269 40 74

Email: espana@clinisciences.com Web: https://www.clinisciences.com

Company: CliniSciences Limited Address: 11 Progress Business center, Whittle Parkway, SL1 6DQ Slough- United Kingdom

Telephone: +44 (0)1753 866 511 or +44 (0) 330 684 0982 Fax: +44 (0)1753 208 899

Email: uk@clinisciences.com IWeb: https://www.clinisciences.com

### Denmark

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: danmark@clinisciences.com Web: https://www.clinisciences.com

### Germany

Company: Biotrend Chemikalien GmbH Address: Wilhelm-Mauser-Str. 41-43,

50827 Köln - Germany Telephone: +49 221 9498 320 Fax: +49 221 9498 325 Email: info@biotrend.com Web: https://www.biotrend.com

## Italy

Company: CliniSciences S.r.I Address: Via Maremmana inferiore 378 Roma 00012 Guidonia Montecelio - Italy

Telephone: +39 06 94 80 56 71 Fax: +39 06 94 80 00 21 Email: italia@clinisciences.com Web: https://www.clinisciences.com

# Poland

Company: CliniSciences sp.Z.o.o. Address: ul. Rotmistrza Witolda Pileckiego 67 lok. 200 - 02-781 Warszawa -Poland

Telephone: +48 22 307 0535 Fax: +48 22 307 0532

Email: polska@clinisciences.com Web: https://www.clinisciences.com

# Sweden

Company: CliniSciences ApS Address: Oesterbrogade 226, st. 1, Copenhagen, 2100 - Denmark Telephone: +45 89 888 349 Fax: +45 89 884 064

Email: sverige@clinisciences.com Web: https://www.clinisciences.com

Company: Biotrend Chemicals LLC Address: c/o Carr Riggs Ingram, 500 Grand Boulevard, Suite 210 Miramar

Beach, FL 32550- USA Telephone: +1 850 650 7790 Fax: +1 850 650 4383

Email: info@biotrend-usa.com Web: https://www.biotrend-usa.com











